

Bennett Security Limited

Lynkyla, Old Inn Road, Findon, Aberdeen, AB12 3RL

Tel 01224 783195 Mobile 07977 119402

E-mail: mike@bennettsecurity.co.uk

www.bennettsecurity.co.uk

CUSTOM MADE NURSE CALL AIDS

Wireless Infra Red Movement Sensor

This sensor is designed to alert staff of movement within a room fitted with an Intercall infra red nurse call point. It can be positioned on the floor or on bedside furniture. It is self resetting - you need only reset the nurse call system.

Instructions

Switch the unit on and point it in the direction of the area to be monitored. Allow 30 seconds for the unit to settle. The red indicator will light on movement and aid the correct positioning. Turn on the "alert" switch and any further movement will trigger the nurse call point. Switch off when not in use. Test regularly to ensure correct operation.

Changing Batteries

Remove the lid with 4 screws, carefully prise out the batteries and replace with new ones. The batteries fitted have been chosen for their high capacity: other AA batteries will work but with reduced life.

Technical Specification

- Detection Angle: 100°
- Detection Range: 6-8 metres
- Batteries: 4 x AA Energizer Lithium and 1 x LR23 transmitter
- Output: Infra red transmission
- Working Temperature: -10°C to +40°C
- Working Humidity: < 93 % RH
- Dimensions: 150 x 65 x 40 mm
- Weight: ± 250g

If you do not have infra red call points, they can be easily fitted in place of existing call points which will also allow the use of neck pendants. (Infra red call points can be identified by a large black dot between the call & reset buttons)

These movement sensors will be covered by a one year guarantee against defective materials or workmanship. It does not apply in cases of damage, wear and tear, unauthorised modification or adjustment.

Feedback and suggestions for improvement are very welcome.